



## Integrity® Static Cushion range



**We asked you what you needed to switch to INTEGRITY® as your first choice in cushions. You said you wanted:**

1. A full range, covering different risk levels and weight limits
2. Clearly labelled
3. Non-slip base
4. A zip on two sides
5. Different cover options - black is the best colour
6. Special sizes
7. Easy to order
8. Quick delivery
9. Available in a box with a handle or a polythene bag with a handle
10. Affordable

☐  
☐  
☐  
☐  
☐  
☐  
☐  
☐  
☐  
☐  
☐  
☐

*Sumed has 35 years' experience in selling cushions, but we are seen as a supplier of speciality products. With the new Integrity® range, we are looking to change this*

With outstanding performance but competitively priced, Sumed's comprehensive Integrity® range of static cushions is the perfect choice whenever it is essential to minimise localised pressure. They are highly comfortable, lightweight, low maintenance and ideal for use with chairs, wheelchairs and in most other seating situations. The range covers 5 levels of pressure care: ENTRY LEVEL, LOW RISK, MEDIUM RISK, HIGH RISK and VERY HIGH RISK, come with a choice of 3 cover types: SPACER FABRIC, TOWELLING and WIPEABLE, and are readily available in a range of sizes and user weights.

With so many options available, you're sure to find an Integrity® cushion that will suit your individual needs.

# THE CLEAR CHOICE

## With 5 levels of pressure care

At Sumed®, we understand the importance of clear product labelling, both in terms of safety, i.e. to ensure that the right product is being used, and for convenience, i.e. to easily identify stock. That is why we've implemented a simple colour system for the labelling of our Integrity® range of static cushions.

To make risk level identification easier and quicker to determine, each level has been allocated a colour and this is indicated on the cushion by a label clearly displayed in that colour. In addition, the label also denotes the risk level with a single letter: E = Entry Level, L = Low Risk, M = Medium Risk, H = High Risk and V = Very High Risk. The maximum user weight for the cushion is also shown.



### The risk levels are:

- ENTRY LEVEL** – For Comfort Only
- LOW** – Low Risk of Pressure Damage
- MEDIUM** – Medium Risk of Pressure Damage
- HIGH** – High Risk of Pressure Damage
- VERY HIGH** – Very High Risk of Pressure Damage

The label is positioned on the front right of the cushion and on all packaging, which is the optimum place for it to be seen when stocked, i.e. stacked on shelves, or transported. When a cushion is in use, the label, though covered by the user's left leg, can be easily referenced should the cushion type need to be identified.

This simple colour system combined with proper labelling to clearly establish a difference in risk level not only makes storage, stock-taking, ordering and delivery more efficient, but it also greatly reduces the chances of a cushion being used at the wrong risk level and/or weight limit.

COLOUR SIGNIFIES  
RISK LEVEL

LETTER SIGNIFIES  
RISK LEVEL



**MAXIMUM WEIGHT LIMIT** The maximum weight limits are:

**17 stone** (108kg) – All risk levels

**28 stone** (178kg) – Not Very High Risk

**50 stone** (318kg) – Not Entry Level, High or Very High Risk


- Pressure care rating at 5 levels
- Easy to differentiate between risk levels at a glance
- Maximum user weight clearly displayed on the label
- Cushion label is easily seen when stocked (i.e. when stacked)
- Cushion label is hidden, yet easily referenced, when in use



# 3 COVER OPTIONS

To suit every need

**Sumed has traditionally offered cushions only in a wipe-down option, but research has guided us into introducing two additional cover options: towelling and spacer fabric.**

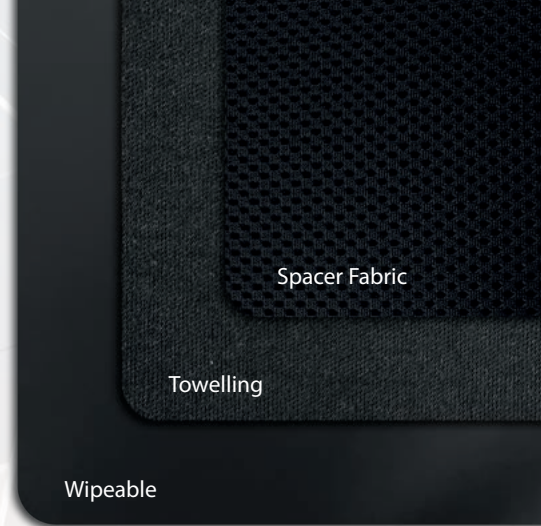

## WIPEABLE

The wipeable cover is a four way stretch weft knitted nylon fabric coated with breathable polyurethane. This fabric can be washed at high temperatures and is ideal when there is a likelihood of fluid ingress into the insert. It has flame retardant properties.



## TOWELLING

The towelling fabric option is a four way stretch knitted structure with inherent flame retardant properties but being constructed from a blend of modacrylic, cotton and carbon. The blend of cotton and acrylic give a very soft and absorbent feel and the cover is very lightweight.



## SPACER FABRIC

The spacer fabric is manufactured from 100% polyester and has a combination of monofilaments to provide cushioning and textured multifilaments to give a soft comfortable feel. The construction is flame retardant and provides a layer of air which allows heat and moisture vapour to be dissipated thus providing a very comfortable surface.



Non-slip base



Packaging option:  
Box with handle



Packaging option:  
Polythene bag with handle

- Cushion inserts are engineered to meet the risk/weight requirements
- Warranty 3 years *(against manufacturing defects)*
- Prescription forms are available to make placing an order even easier

1. A full range, covering different risk levels and weight limits
2. Clearly labelled
3. Non-slip base
4. A zip on two sides
5. Different cover options - black is the best colour
6. Special sizes
7. Easy to order
8. Quick delivery
9. Available in a box with a handle or a polythene bag with a handle
10. Affordable



*We listened and ticked all the boxes!*



Sumed® have been manufacturing and supplying pressure relief cushions of the highest standard for over 35 years. Our comprehensive range of speciality cushions, which includes medical, wheelchair, commode and bath cushions, not only meets the market demand at every level, but exceeds it.

**To find out more, contact us today**

**Tel: 01457 890980**

**Email: [sales@sumed.co.uk](mailto:sales@sumed.co.uk)**

**[www.sumedinternational.com](http://www.sumedinternational.com)**

**Sumed International (UK) Ltd.**

Integrity House, Units 1 -2 Graphite Way  
Hadfield, Glossop  
Derbyshire SK13 1QH

